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Sign in

Don't have an account? [Sign up](#)

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The screenshot displays the ElevenLabs Creative Platform interface. On the left, a sidebar lists various tools and products. The 'Developers' option at the bottom of the sidebar is highlighted with a red arrow. The main content area shows the 'Developers' section, which includes tabs for 'Overview', 'API Keys', and 'Web'. Below the tabs, there is a 'Developer quicks' section with a 'Get started' button and a 'Quick Links' section with buttons for 'Create an API Key' and 'Libraries & SDKs'.

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The screenshot shows the 'Developers' section of the ElevenLabs website. At the top, there are navigation links for 'Overview', 'API Keys', 'Webhooks', and 'Usage', along with buttons for 'API Pricing' and 'Documentation'. Below this is a 'Developer quickstart' section with a 'Get started' button and a code snippet in JavaScript. The code snippet is as follows:

```
1 import { ElevenLabsClient, play } from '@elevenlabs/elevenlabs-js';
2
3 const elevenlabs = new ElevenLabsClient({
4   apiKey: "YOUR_API_KEY", // Defaults to process.env.ELEVENLABS_API_KEY
5 });
6
7 const audio = await elevenlabs.textToSpeech.convert(
8   'JBFqnCBsd6RMkjVDRZzb', // voice_id
9   {
10    text: 'The first move is what sets everything in motion.',
11    modelId: 'eleven_multilingual_v2',
12    outputFormat: 'mp3_44100_128', // output_format
13  }
14 );
15
16 await play(audio);
```


Below the code snippet is a 'Quick Links' section with five buttons: 'Create an API Key' (highlighted with a red box), 'Browse Models', 'API Reference', 'Libraries & SDKs', 'ElevenLabs Agents', and 'Pricing Overview'.

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The screenshot shows the 'User API Keys' section of the ElevenLabs website. At the top, there are navigation links for 'Overview', 'API Keys', 'Webhooks', and 'Usage', along with buttons for 'API Pricing' and 'Documentation'. Below this is the 'User API Keys' section with a description: 'An API key lets you connect to our API and use its features. You can create multiple keys with different access levels.' and a '+ Create Key' button.

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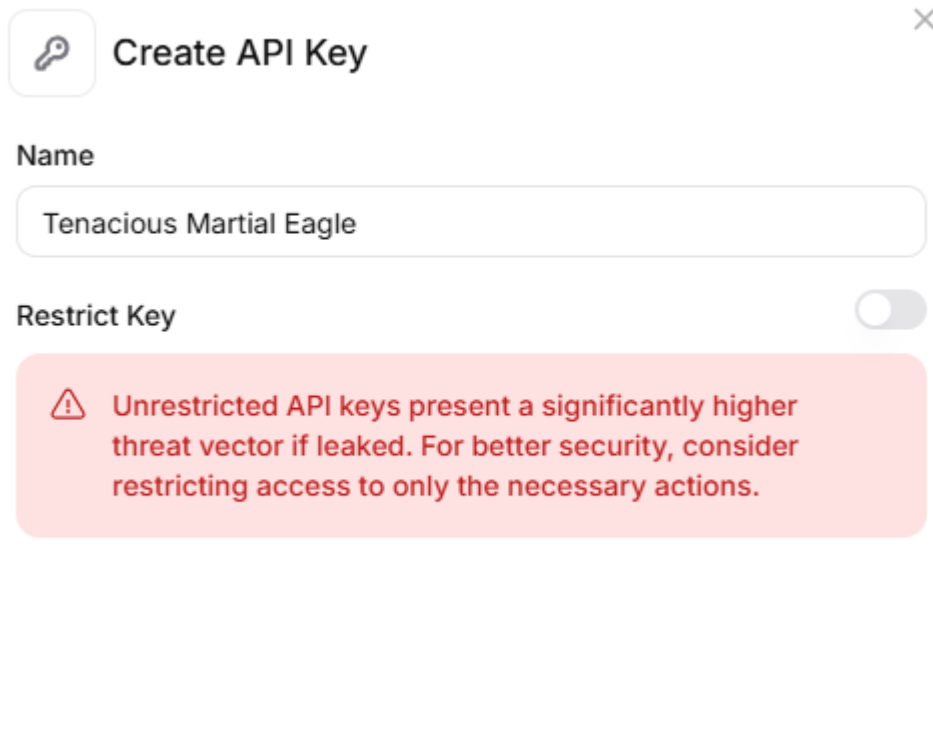
[Documentation ↗](#)

User API Keys

An API key lets you connect to our API and use its features. You can create multiple keys with different access levels.

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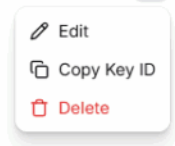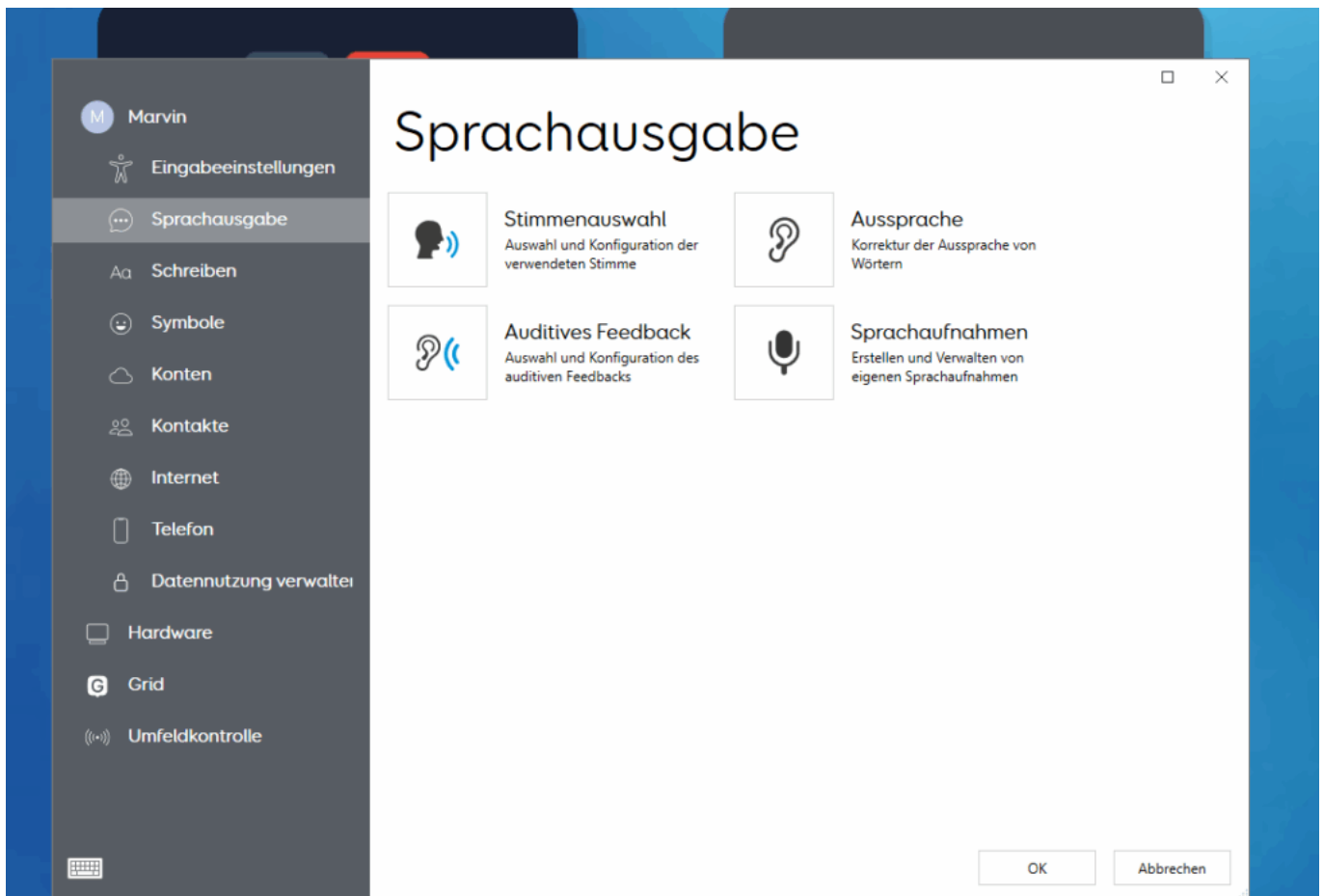
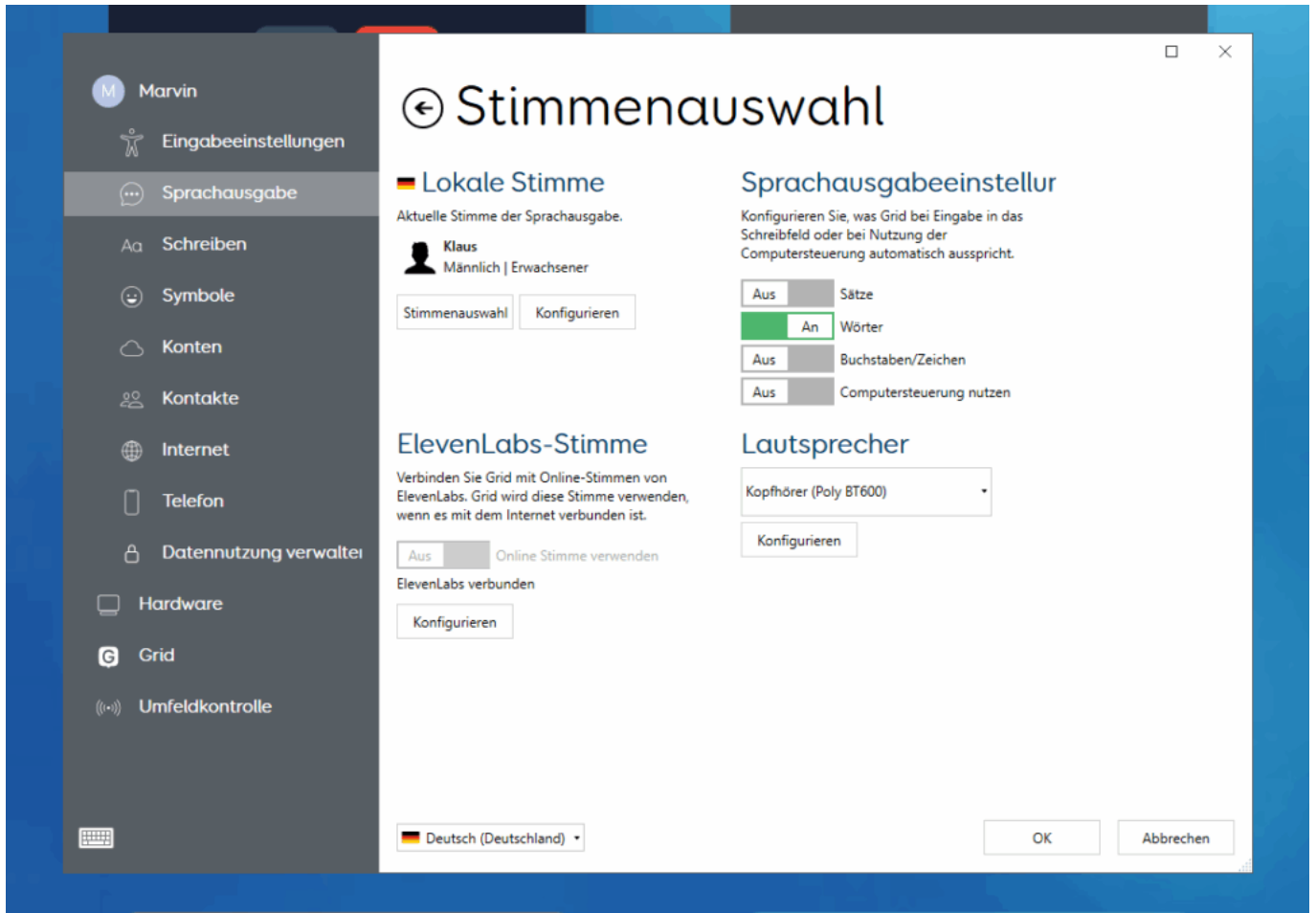
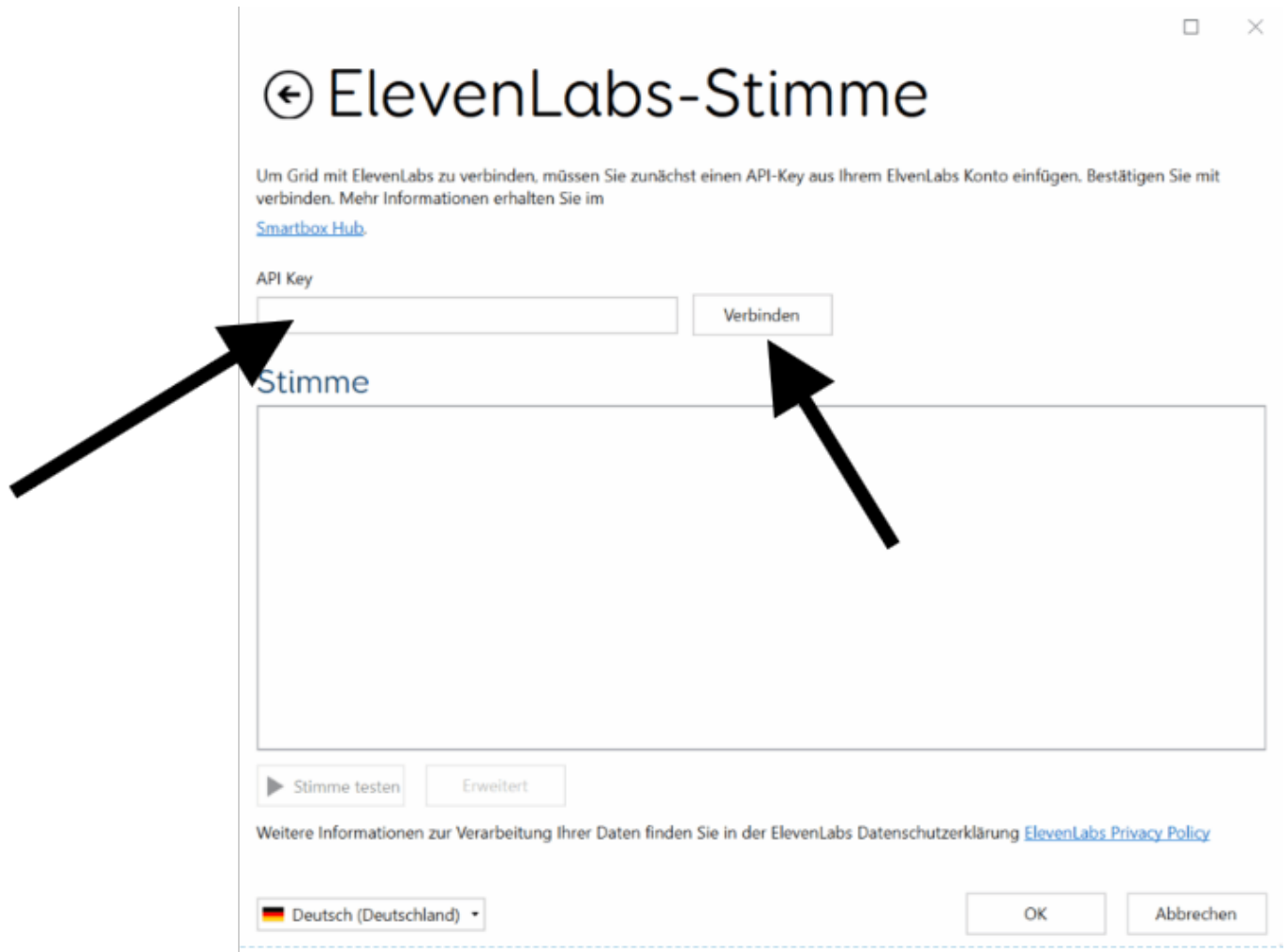
Environment control

ElevenLabs

To connect Grid to ElevenLabs you must first provide an API key. For information on how to find this, please visit the [Smartbox Hub](#).

API key

Disconnect ✓

Voice

- Tizzy Instant voice - Cloned
- Tizzy Instant voice 2 - Cloned
- Audiobook voice - Generated
- Cheery voice design - Generated
- Voice design male - Generated
- Alice - Premade
- Aria - Premade
- Bill - Premade

Listen Advanced

Credits: 33,258 / 110,000

For information on data processed by ElevenLabs, see [ElevenLabs Privacy Policy](#)

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